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Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

Ascensia Diabetes Care Italy S.r.l.
Via Varesina, 162
20156 Milano, Italy

ASCENSIA
Diabetes Care

© 2024 Ascensia Diabetes Care Holdings AG.
Toate drepturile rezervate.

90012826
Rev. 10/24

Need Help?

For more information, go to www.diabetes.ascensia.com

Your meter is preset and locked to display results in mg/dL. If your result is in mmol/L, contact Customer Service.

Imported and Distributed by:
Unipharm Trading LLC
7th Floor, KIA Motors Showroom,
PO Box 6255, Al Rayyan Road, Doha, Qatar.
+974 44930058 / +974 44631647
info@unipharm.com.qa
www.unipharm.com.qa
Scientific & Technical Services Co. WLL
Office 2, 2nd Floor, Building 271, St. 230, Zone 41,
Financial Square, C. Ring Road, Doha, Qatar.
+974 44621899 / +974 77695520 / +974 55316076
mail@stsqatar.com

For in vitro diagnostic use.

Ascensia, the Ascensia Diabetes Care logo, Contour, Microlet, the No Coding logo, Second-Chance, the Second-Chance sampling logo, Smartcolour, and Smartlight are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Ascensia Diabetes Care is under license.

Google Play and the Google Play logo are trademarks of Google LLC.

All other trademarks are properties of their respective owners. No relationship or endorsement should be inferred or implied.

Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

Ascensia Diabetes Care Italy S.r.l.
Via Varesina, 162
20156 Milano, Italy

ASCENSIA
Diabetes Care

© 2024 Ascensia Diabetes Care Holdings AG.
All rights reserved.

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Contour
plus ELITE
Blood Glucose Monitoring System

Works with free CONTOUR®DIABETES app.

To check meter online registration availability, visit
www.diabetes.ascensia.com/registration

Uses only CONTOUR®PLUS
blood glucose test strips.

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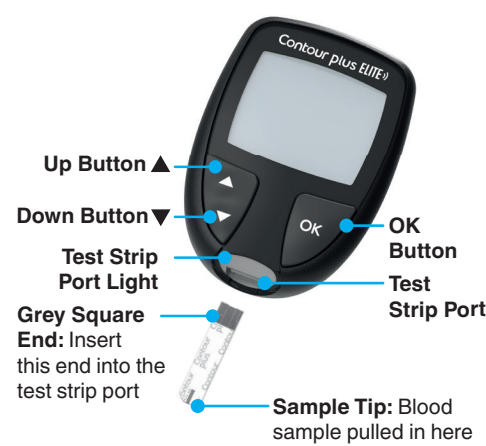
Your CONTOUR PLUS ELITE meter

NOTE: Your meter comes with a pre-set time, date, and target range. You can change these features in **Settings**.

NOTE: Your CONTOUR PLUS ELITE meter works only with CONTOUR PLUS test strips and CONTOUR PLUS control solution.

Read your CONTOUR PLUS ELITE user guide for the **Intended Use** of the meter and a list of **Important Safety Information**, Warnings, Limitations, and Precautions.

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9

Settings (continued)

3. To scroll through Settings options as shown in the image, press the ▲ or ▼ button until the symbol you want is **blinking**.
4. Press the **OK** button to select the blinking symbol.
5. To edit a Setting, press the ▲ or ▼ button. You can turn the Setting On or Off, or change the value.
6. To accept the change and return to the **Home** screen, press the **OK** button.

To exit from Settings and return to Home:

1. Press the ▲ or ▼ button until the **Settings** symbol is blinking.
2. Press the **OK** button. If you are in a Setting such as Date and you need to exit, press the **OK** button repeatedly until you get back to the **Home** screen.

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Set Overall Target Range

WARNING

Talk to your health care professional before setting any **Target Ranges** in your meter or in compatible Ascensia Diabetes Care software.

If **Meal Markers** are Off, you can set only 1 **Target Range**. To set **Before Meal** and **After Meal Target Ranges**, see the CONTOUR PLUS ELITE user guide.

1. To highlight the **Settings** symbol from the **Home** screen, press the ▼ button.
2. When the **Settings** symbol is blinking, press the **OK** button to enter **Settings**.
3. To scroll to the **Target Range** symbol, press the ▲ or ▼ button. When the symbol is blinking, press the **OK** button.
4. To set the blinking **Overall Low Target**, press the ▲ or ▼ button, then press the **OK** button.

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11

5. To set the blinking **Overall High Target**, press the ▲ or ▼ button, then press the **OK** button.

Turn Meal Markers On

1. To highlight the **Settings** symbol from the **Home** screen, press the ▼ button.
2. When the **Settings** symbol is blinking, press the **OK** button to enter **Settings**.
3. To scroll to the **Meal Marker** symbols, press the ▲ or ▼ button.
4. When the **Meal Marker** symbols are blinking, press the **OK** button.
5. Press the ▲ or ▼ button until On is **blinking**, then press the **OK** button.

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12

NOTE: With **Meal Markers** On, you can set **Before Meal** and **After Meal Target Ranges** (see the CONTOUR PLUS ELITE user guide). You can also set a **Reminder**.

Turn Reminder Feature On

When **Meal Markers** are On, you can set a **Reminder** to test your blood glucose after you mark a reading as **Before Meal**. If **Meal Markers** are Off, see *Turn Meal Markers On* in this guide.

1. To highlight the **Settings** symbol from the **Home** screen, press the ▼ button.
2. When the **Settings** symbol is blinking, press the **OK** button to enter **Settings**.
3. To scroll to the **Reminder** symbol, press the ▲ or ▼ button.
4. When the **Reminder** symbol is blinking, press the **OK** button.
5. Press the ▲ or ▼ button until On is blinking, then press the **OK** button.

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13

Select a Meal Marker During a Blood Glucose Test

Meal Marker Symbols

- Fasting
- After Meal
- Before Meal
- No Mark

NOTE: You must do a blood glucose test before you can select a **Meal Marker**. **You cannot select a Meal Marker in Settings**.

1. Test your blood glucose. See *Fingertip Testing* in this guide.
2. Do not remove the test strip until you select a **Meal Marker**. A **Meal Marker** blinks on the blood glucose result screen.
3. If the blinking **Meal Marker** is the one you want, press the **OK** button. To select a different **Meal Marker**, press the ▲ or ▼ button.
4. When the **Meal Marker** you want is **blinking**, press the **OK** button.

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14

NOTE: If you do not select a **Meal Marker** within 3 minutes, the meter turns Off. Your blood glucose reading is stored in the **Logbook** without a **Meal Marker**.

Set a Reminder

1. Make sure the **Reminder** feature is On in **Settings**.
2. Mark a blood glucose reading as a **Before Meal** reading, then press the **OK** button.
3. To scroll from 2 hours to 0.5 hours by half-hour intervals, press the ▲ or ▼ button.
4. To set the **Reminder**, press the **OK** button.

The screen returns to the **Before Meal** reading. The **Reminder** symbol displays to confirm the Reminder is set.

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15

LO Results

If the meter beeps twice and displays the **LO** screen, your blood glucose reading is under 10 mg/dL.

Follow medical advice immediately. Contact your health care professional.

HI Results

If the meter beeps once and displays the **HI** screen, your blood glucose reading is over 600 mg/dL.

1. Wash and dry your hands well.
2. Retest with a new strip. If results are still over 600 mg/dL, **follow medical advice immediately**.

To turn the meter Off, remove the test strip.

WARNING

- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- Health care professionals should follow the biohazard disposal requirements for their facility.

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16

Your Meter Display

| Symbol | What It Means |
|--------|--|
| | Your Logbook |
| | Meter Settings |
| | smartLIGHT target range indicator (Yellow=Above Target; Green=In Target; Red=Below Target) |
| | A Target Range or Target Range setting |
| | Blood glucose test result is above Target Range |
| | Blood glucose test result is in Target Range |
| | Blood glucose test result is below Target Range |

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17

| Symbol | What It Means |
|--------|--|
| | Fasting marker; used when testing after fasting (no food or drink for 8 hours, except water or non-caloric beverages) |
| | Before Meal marker; used when testing within 1 hour before a meal |
| | After Meal marker; used when testing within 2 hours after the first bite of a meal |
| | No Meal marker selected |
| | Reminder feature |
| | Sound feature |

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18

| Symbol | What It Means |
|--------|---|
| | Indicates the Bluetooth wireless setting is On; the meter can communicate with a mobile device |
| | Indicates a meter error |
| | Batteries are low and need to be changed |
| | 7-, 14-, 30-, and 90-day averages |
| | Total number of blood glucose readings used to calculate averages |
| | Control solution test result |
| | Add more blood to same test strip |

The CONTOUR DIABETES app allows you to transfer results from your meter to your compatible smartphone or tablet. See the meter user guide for instructions and features.

18